

Pre-Installation Checklist



EVA VET WITH VIEWALL 3.2 SOFTWARE PRE-INSTALLATION CHECKLIST

Required:

- Be sure to have in-clinic IT support available for the installation
- Identify, locate and specify the HOST/SERVER computer
 - Physical location
 - Windows computer name or IP address
 - Operating system
 - Administrator account name and password
- Identify, locate, count and specify all client workstation computers
 - Physical location(s)
 - Windows computer name(s) and/or IP address
 - Operating system(s): Compatible with Windows 7 & Windows 8 32/64Bit
 - Administrator account name and password
- Verify that all the target computers meet the minimum requirements:
<http://www.imageworksveterinary.com/evavet>
- Identify any firewalls and antivirus/malware software used on computers or the local network
 - Have the administrator passwords available if needed
- If you will be bridging to Practice Management Software, identify and have the PM SW supplier's contact information available
- If connecting ViewAll to other manufactures' hardware (e.g. intra-oral video cameras, TWAIN compatible x-ray scanners, etc.), please have the other manufacturers' contact information available
- Internet access to the computers (for registration and if necessary remote diagnostics)

Recommended:

- An extra USB powered hub that plugs into the wall
- An extra USB Cable

A typical installation will take 15 to 20 minutes per computer and about 25-30 minutes for a network install of up to five computers.



Installation Record

(Make copies for more computers if necessary and file)

Practice Name: _____ **Contact Name:** _____

Main Phone Number: _____ **Email:** _____

Address: _____

Computer Type:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:			IP Address:
Operation System:			Service Pack:
Physical Location:			
ViewAll Version:			Date:
IWV Hardware Connected:	<input type="checkbox"/> None <input type="checkbox"/> EVA Vet		
Other Hardware Connected: <i>(Specify type, make, and model)</i>			

Computer Type:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:			IP Address:
Operation System:			Service Pack:
Physical Location:			
ViewAll Version:			Date:
IWV Hardware Connected:	<input type="checkbox"/> None <input type="checkbox"/> EVA Vet		
Other Hardware Connected: <i>(Specify type, make, and model)</i>			

Computer Type:	<input type="checkbox"/> Server	<input type="checkbox"/> Client	<input type="checkbox"/> Standalone
Name:			IP Address:
Operation System:			Service Pack:
Physical Location:			
ViewAll Version:			Date:
IWV Hardware Connected:	<input type="checkbox"/> None <input type="checkbox"/> EVA Vet		
Other Hardware Connected: <i>(Specify type, make, and model)</i>			